



ACCURE

All patients should be carefully screened prior to Accure laser treatment.

Contraindications include:

- Cancer; in particular, skin cancer
- Pregnancy
- Use of photosensitive medications and herbs for which 300 to 2940nm light exposure is contra-indicated
- Diseases which may be stimulated by light at 300 to 2940nm
- Prolonged exposure to sun or artificial tanning during the 3 to 4 weeks prior to treatment and post treatment
- History of keloids or hypertrophic scarring
- Active infection of herpes simplex in the treatment area
- Diabetes (insulin dependent)
- Fragile and dry skin
- Hormonal disorders (that are stimulated under pulsed light)
- Use of anticoagulants
- Epilepsy
- History of coagulopathies
- Immune deficiency disease or an immuno-compromised status
- Oral use of retinoids (Accutane)
- Oral use of steroids

Additional insights for patient screening:

- Ongoing therapies or therapies carried on in the past based on drugs containing metals or other substances that could absorb laser radiation must be performed under the physician or operator responsibility after a careful evaluation and verification (e.g., gold-based drugs for the treatment of arthritis).
- Any skin resurfacing procedures, including ablative and non-ablative energy-based device procedures, chemical peels, and any other procedures that affect the epidermis or dermis performed before the use of Accure device shall be evaluated by the physician. Skin should be completely healed from any former treatment. Do not expose irritated or breached skin to Accure laser.

Potential Side Effects

- Tissue damage, severe bleeding
- Discomfort
- Damage to Natural Skin Texture
- Change of Pigmentation
- Scarring
- Excessive Swelling
- Fragile Skin
- Bruising

Potential Residual Risks

- Severe or prolonged erythema (redness) and edema (swelling)
- Irritation, itching, a mild burning sensation or pain
- Blisters, epidermal erosions, or peri-lesional hyperpigmentation
- Premature aging
- Eye damage from reflected or prolonged unprotected exposure to laser light.
- Thermal damage and unwanted adverse effects

Warnings Related to the Applications

- The laser device should be applied for treatment to intact skin only without the presence of tattoos
- The application point should not be close to the outer canthus of the eye
- Appropriate safety eyewear should always be worn by the patient when treating with the Accure laser, including when treating near the eye
- If bleeding occurs at any time during the use of the Laser Device, discontinue use immediately and isolate the handpiece